


PATRAN[®]app

PATRAN[®]app Instructions


Use PATRAN[®]app to apply closed- or open-toe compression hose on a patient who is seated or lying in bed.

APPLY COMPRESSION HOSE

- 1** PATRANapp is a large loop. Insert the foot and leg into the loop as shown. Make sure the loop is clear of the toes. If possible, have the patient hold the top of the loop, so the PATRANapp stays in place. The caregiver should be next to and facing the same direction as the patient to pull rather than push up the hose.

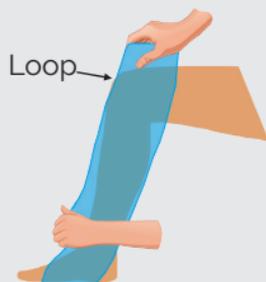

- 2** Pinch the loop closed in front of the ankle and pull the hose over the patient's heel. If hose extends to thigh, gather it above the heel until it fits foot correctly.

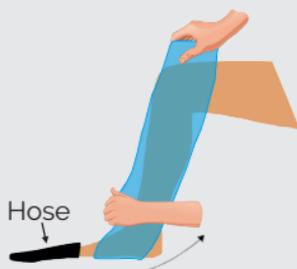

- 3** Adjust the PATRANapp so it covers as much surface of the leg as possible. Then pull up hose.



REMOVE LOOP

- 4** Grab the sides of the loop – one hand on each side of the knee.
- 5** Pull both sides of the loop behind the knee.
- 6** Pull on each side of the loop in a sawing motion until it slips off the heel. Then pull the loop out of the hose.



PATRAN[®]app

PATRAN[®]app Instructions

Use PATRAN[®]app to remove closed- or open-toe compression hose on a patient who is seated or lying in bed.


REMOVE HOSE

1 If hose extends to thigh, pull top of hose down to the knee.

2 Wrap the PATRANapp loop around back of the leg near the ankle so it is covering the calf and heel.



3 Hold the loop in place in front of the leg with one hand. Reach behind the knee and pull down only on hose with other hand. Release the loop as hose pulls over it. You may consider holding the patient's leg after releasing the loop.

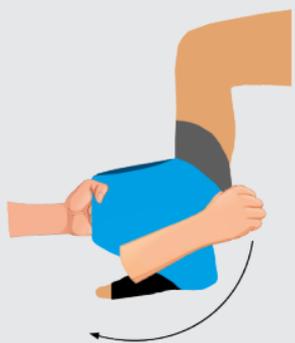

**WATCH THE VIDEO AT
[PATRAN.NET/DOWNLOAD-INSTRUCTIONS.](https://patran.net/download-instructions)**

PATRANapp can be used multiple times with the same patient. Replace when PATRANapp is no longer slippery.

WARNING: Strangulation hazard. Keep loops away from children and anyone with mental health conditions.

JAMAR HEALTH PRODUCTS, INC.

patran.net
patrans@patran.net
414-529-5175